NovaLife TRE™

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Inhoud

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12000222

Dansac A/S
Lyngby Hovedgade 4
2800 Kongens Lyngby, Denmark www.dansac.com

For reference only. Unauthorized use, disclosure or copying is prohibited.

NovaLife TRE™

en • 1-Piece Closed Pouch

INSTRUCTIONS FOR USE

Contents

- Closed pouch with attached TRE skin barrier
- Stoma guide for cut-to-fit products. Use for pre-cut products is optional.

Product Description

Closed pouch with skin barrier designed with TRE pH buffering technology, suitable for standard wear. The pouch is designed to be removed and disposed of after use. The pouch is most suitable for colostomies or ileostomies.

Skin barriers are available in many styles:

- a flat skin barrier is best suited for a protruding stoma.
- a convex or soft convex (curved) skin barrier is best suited for a stoma that does not protrude. Convex pouches have belt tabs (belt not included).
- a cut-to-fit product can be cut by the user to fit the stoma.
- a pre-cut product is provided for convenience.

Made with hydrocolloid skin adhesive, polymer/copolymer plastics, and nonwoven fabric.

Indications and Intended Use

This product is intended to collect output from a stoma (e.g., colostomy, ileostomy) or skin opening (e.g., wound, fistula).

WARNINGS

Reuse of a single use product may create potential risk to the user such as poor adhesion. Reprocessing, cleaning, disinfection, and sterilisation may compromise the product. This may result in odour or leakage. Ensure proper fit as instructed. If the barrier opening is incorrect or leakage occurs, skin damage may occur (e.g., irritation, broken skin, rash, or redness). This may decrease wear time.

Notes:

To help ensure proper adhesion, apply to clean, dry intact skin. Do not use oils or lotions. These products may interfere with adhesion of the skin barrier. Product fit is affected by many factors such as stoma construction, stoma output, skin condition, and abdominal contours. Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.dansac.com/authority or contact Authorised Representative /Responsible Person or local distributor.

 	
CE marked as required for EU markets	
UKCA marked as required for UK market	
Not made with natural rubber latex.	

 	
MD Medical Device	
(as applicable).	

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