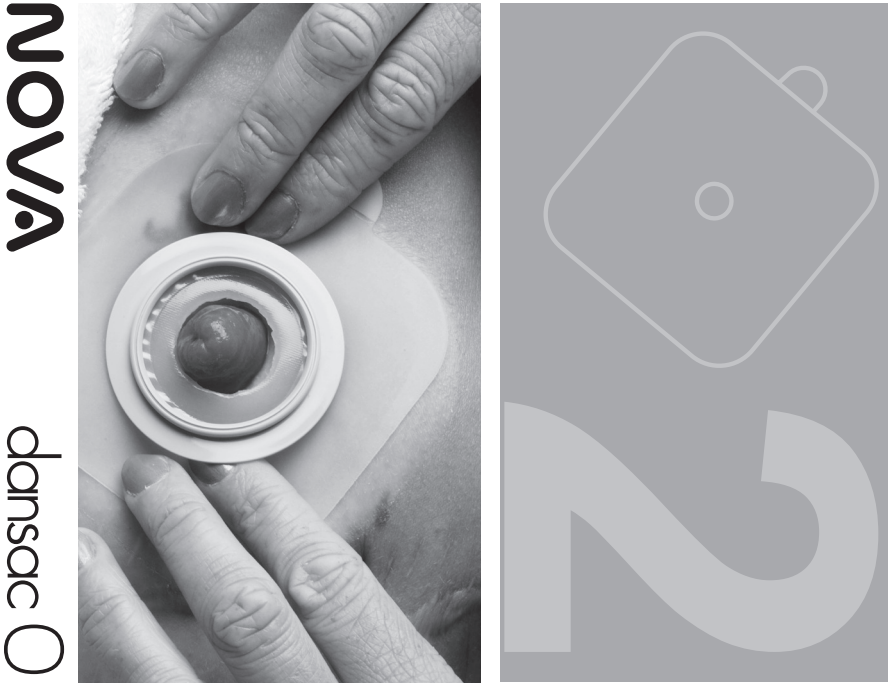
en • 2 Piece Skin Barrier

INSTRUCTIONS FOR USE

Contents

- Skin barrier with attached coupling ring
- Stoma guide for cut-to-fit products. Use for pre-cut products is optional.

Product Description

This skin barrier is suited for frequent pouch changes or use on fragile skin. Designed to be used with a corresponding pouch with the same ring size.

Skin barriers are available in many styles:

- a flat skin barrier is best suited for a protruding stoma.
- a convex or soft convex (curved) skin barrier is best suited for a stoma that does not protrude.
- a cut-to-fit product can be cut by the user to fit the stoma.
- a pre-cut product is provided for convenience.

Made of hydrocolloid skin adhesive and polymer/copolymer plastics.

Indications and Intended Use

This product is part of a two-piece ostomy pouching system intended to collect output from a stoma (e.g., colostomy, ileostomy, or urostomy) or skin opening (e.g., wound, fistula).

WARNINGS

Reuse of a single use product may create potential risk to the user such as poor adhesion. Reprocessing, cleaning, disinfection, and sterilisation may compromise the product. This may result in odour or leakage. Ensure proper fit as instructed. If the barrier opening is incorrect or leakage occurs, skin damage may occur (e.g., irritation, broken skin, rash, or redness). This may decrease wear time.

Notes:

To help ensure proper adhesion, apply to clean, dry intact skin. Do not use oils or lotions. These products may interfere with adhesion of the skin barrier. Product fit is affected by many factors such as stoma construction, stoma output,skin condition, and abdominal contours. Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.dansac.com/authority or contact Authorised Representative /Responsible Person or local distributor.

CE marked as required for EU markets
UKCA marked as required for UK market
Not made with natural rubber latex.

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