12000329

www.dansac.com/contact

www.dansac.com/symbolglossary

Dansac A/S
Lynby Hovedgade 4
2800 Kongens Lyngby, Denmark www.dansac.com

dansac

12000329-1025 ©2025 Dansac A/S

en • Dansac Seals

Contents: Ostomy Seals

Product Description: Ostomy seals designed to improve the fit of an ostomy pouching system. Made with hydrocolloid skin adhesive.

Indications and Intended Use: This product is an accessory to an ostomy pouching system intended to help prevent stoma output from getting under the skin barrier. Seals may help provide a secure seal under ostomy products.

WARNINGS: Reuse of a single use product may create potential risk to the user such as poor adhesion. Reprocessing, cleaning, disinfection, and sterilisation may compromise the product. This may result in odour or leakage.

Notes: The product may be cut, stretched, or stacked together to help improve fit of a pouching system. Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.dansac.com/authority or contact Authorised Representative/Responsible Person or local distributor.

CE marked as required for EU markets

UKCA marked as required for UK market

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