en• Stoma Irrigation Set

REF	Contents	
Irrigation Set content may vary per market.		
Irrigation Set Products		
09420–0000	Dansac toilet bag	
95200–0000	Water container	
95225–0000	Tubing with connector and regulation clamp	
95205–0000	Stoma cone with tubing	
95220–0000	Cleaning brush	
09547–0000	Silicone ring	
950–20	Irrigation sleeve with ring holder for silicone ring	
R4012–2	Pegs	
950–35	Irrigation sleeve with adhesive skin barrier	
Additional Products		
09000–0000	Ostomy belt	
70005–0001	Dansac Skin Lotion	
71000–0000	Dansac Skin Lotion Tissues	
903–10	Dansac Novalife™ 1-Piece closed pouch	
829–20	Dansac Nova™ Mini Cap	

www.dansac.com/symbolglossary

Product Description

Products to support colostomy irrigation. The irrigation sleeve is designed to be adhered to the skin around the stoma or held in place with a ring and ostomy belt to help direct stoma output into toilet. The water container with stoma cone and tubing are used to flow water into the stoma. The pegs may be used to help close the irrigation sleeve. The cleaning brush is used when rinsing the cone and tubing. Devices are made of polymer/copolymer plastics, silicone, and may contain metal.

Indications and Intended Use

These products are intended to be used for colostomy irrigation. Irrigation is used to help regulate bowel emptying when recommended by a stoma care nurse or healthcare practitioner.

WARNINGS

Use of extremely hot water may cause injury. Complications may also include non-return of fluid, vasovagal response, and abdominal cramps.

For products indicated as:

	Reuse of a single use product may create potential risk to the user such as poor adhesion. Reprocessing, cleaning, disinfection, and sterilisation may compromise the product. This may result in odour or leakage.
---	---

Precautions

The irrigation kit is not intended for use with a urostomy or ileostomy. Injury and/or infection may result if products are used with a urostomy or ileostomy.

For products indicated as:


	Single patient - multiple use. Rinse with clean water, and wipe dry as needed. Discard when visibly damaged or stained. Sharing the product with multiple users creates risk of infection.
--	--

Notes

Irrigation may not be recommended if you have any of the following conditions: current chemotherapy, radiotherapy, stoma stenosis , large parastomal hernia , stoma prolapse, Crohn's disease, or diarrhoea. Contact a stoma care nurse or healthcare practitioner to decide if irrigation is right for you, and for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.dansac.com/authority or contact Authorised Representative /Responsible Person or local distributor.

Please turn over page to see instructions for use.

CE marked as required for EU markets
UKCA marked as required for UK market
Not made with natural rubber latex.

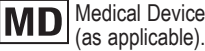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