NovaLife

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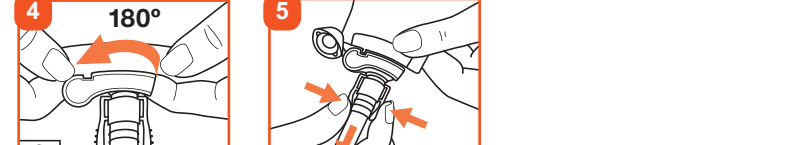
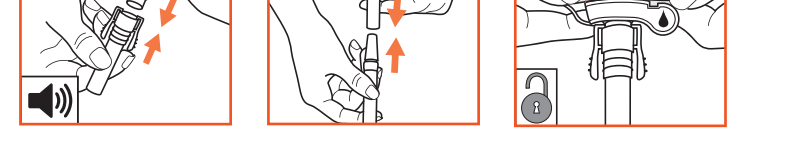
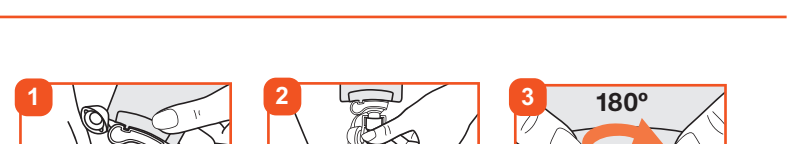
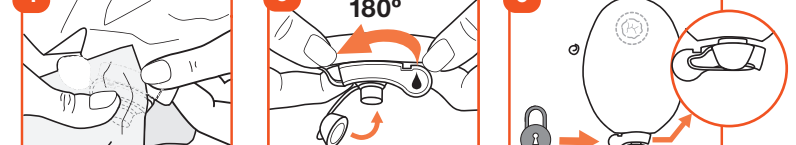
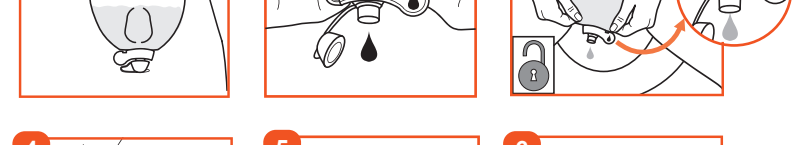
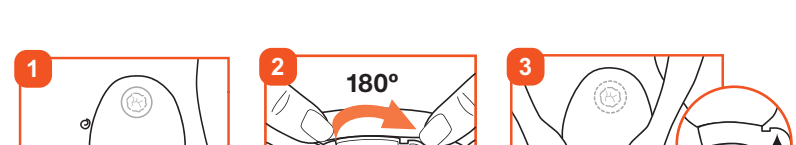
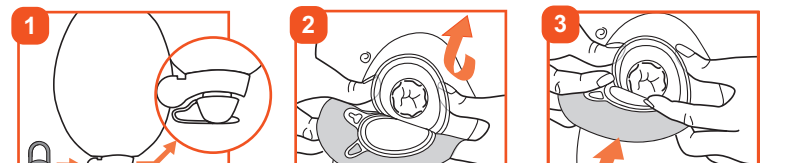
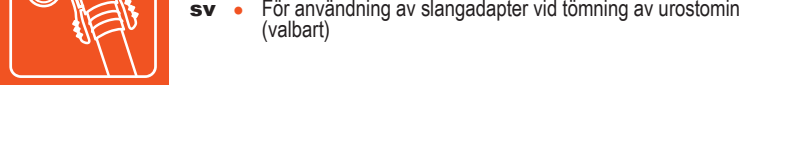
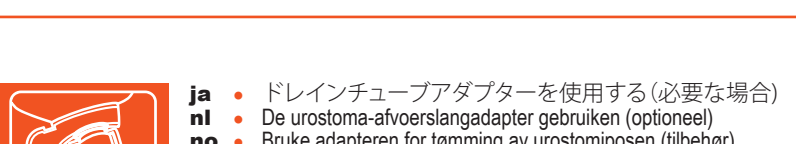
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
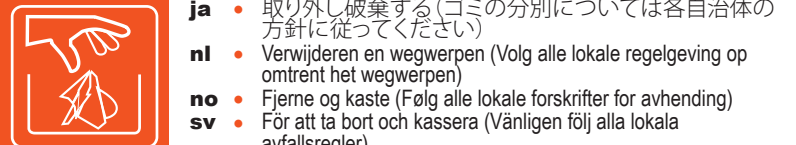
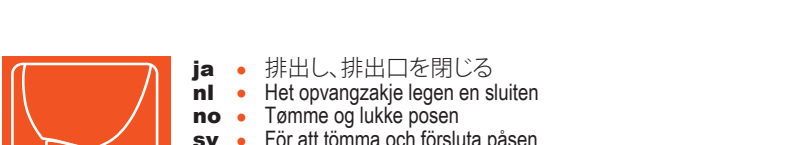
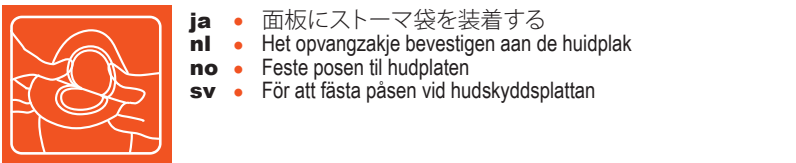
For reference only. Unauthorized use, disclosure or copying is prohibited.

NovaLife

en • 2 Piece Urostomy Pouch

INSTRUCTIONS FOR USE

Contents

- Urostomy pouch with attached coupling ring
 - Ostomy belt tabs (belt not included)
- Urostomy drain tube adapter for use to connect pouch to a drainage bag / leg bag (not included) if desired.

Product Description

Part of a two-piece ostomy pouching system. Designed to be used with a corresponding skin barrier with the same coupling ring size. The pouch is designed to be emptied. The pouch is most suitable for urostomies. It has a tap to empty liquid. Product is made of polymer/copolymer plastics, and nonwoven fabric.

Indications and Intended Use

This product is part of a two-piece ostomy pouching system intended to collect output from a stoma (e.g., urostomy) or skin opening (e.g., wound, fistula).

WARNINGS

Reuse by reprocessing, cleaning, disinfection, and sterilisation may compromise the product. This may result in odour or leakage.

Notes:

Excess mucus from a urostomy stoma may cause the pouch to drain slower than expected. Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.dansac.com/authority or contact local distributor.

Urostomy drain tube adapter

Precaution

 Single patient - multiple use. Rinse with clean water and leave to dry. Discard when visibly damaged. Sharing the urostomy drain tube adapter with multiple users creates risk of infection.

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