nl • Eendelig uro-opvangszakje

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Inhoud

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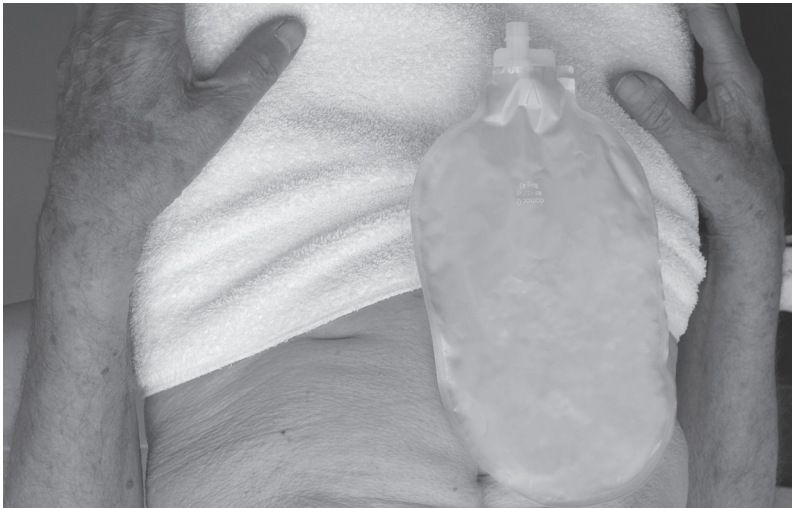
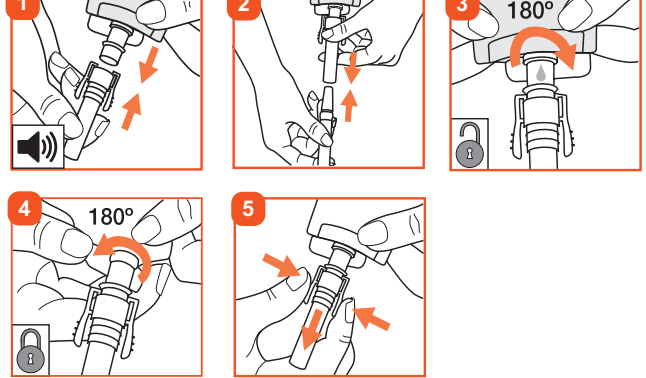
INSTRUCTIONS FOR USE

Contents

- Urostomy pouch with attached skin barrier.
- Urostomy drain tube adapter for use to connect pouch to a drainage bag / leg bag (not included) if desired.
- Plastic caps to cover the tap for a sense of security from drips.
- Stoma guide for cut-to-fit products. Use for pre-cut products is optional.

Product Description

Urostomy pouch with a skin barrier suitable for frequent pouch changes or use on fragile skin. The pouch is designed to be emptied. The pouch is most suitable for urostomies. It has a tap to empty liquid.

Skin barriers are available in many styles:

- a flat skin barrier is best suited for a protruding stoma.
- a convex or soft convex (curved) skin barrier is best suited for a stoma that does not protrude. Convex pouches have belt tabs (belt not included).
- a cut-to-fit product can be cut by the user to fit the stoma.
- a pre-cut product is provided for convenience.

Made with hydrocolloid skin adhesive, polymer/copolymer plastics, and nonwoven fabric.

Indications and Intended Use

This product is intended to collect output from a stoma (e.g., urostomy) or skin opening (e.g., wound, fistula).

WARNINGS

Reuse of a single use product may create potential risk to the user such as poor adhesion. Reprocessing, cleaning, disinfection, and sterilisation may compromise the product. This may result in odour or leakage. Ensure proper fit as instructed. If the barrier opening is incorrect or leakage occurs, skin damage may occur (e.g., irritation, broken skin, rash, or redness). This may decrease wear time.

Notes:

To help ensure proper adhesion, apply to clean, dry intact skin. Do not use oils or lotions. These products may interfere with adhesion of the skin barrier. Excess mucus from a urostomy stoma may cause the pouch to drain slower than expected. Product fit is affected by many factors such as stoma construction, stoma output, skin condition, and abdominal contours. Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.dansac.com/authority or contact local distributor.

Urostomy drain tube adapter

Precaution

 Single patient - multiple use. Rinse with clean water and leave to dry. Discard when visibly damaged. Sharing the urostomy drain tube adapter with multiple users creates risk of infection.

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