
- en • Instructions for use:
To apply
- cs • Návod k použití:
Applikace
- da • Brugsanvisning:
Påsætning
- de • Gebrauchsanleitung:
Anbringen
- fi • Käyttöohjeet:
Levittäminen
- fr • Mode d'emploi :
Mise en place
- hu • Használati útmutató:
Felhelyezéshez
- it • Istruzioni per l'uso:
Come applicare

DANSAC SOFT PASTE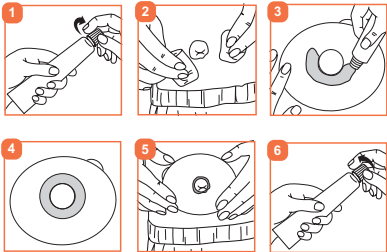

en • Dansac Soft Paste

Contents: Ostomy paste
Product Description: Soft paste is designed to improve the fit of an ostomy pouching system.
Indications and Intended Use: This product is an accessory to an ostomy pouching system intended to help an ostomy product conform to the skin and to help prevent stoma output from getting under the skin barrier.
INGREDIENTS: BUTYL ESTER OF PVM/MA COPOLYMER, ETHYL ALCOHOL, PECTIN, POLYSORBATE 20, SODIUM CARBOXYMETHYLCELLULOSE, POLYETHYLENE FIBERS, SILICA, N-BUTYL ALCOHOL, ALLANTOIN, METHYL PARABEN, BUTYL PARABEN.
Precautions: For external use only.
CAUTION: Contains alcohol. Do not apply to severely irritated or eroded skin, open cuts or sores. A temporary stinging sensation may occur if applied to irritated skin.
Single patient - multiple use
Sharing the product with multiple users creates risk of infection.
Notes: Remove skin barrier release liner before applying paste. Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.dansac.com/authority or contact local distributor.
To dispose. Please follow all local disposal regulations. May use until tube is empty.

Not made with natural rubber latex. **MD** Medical Device (as applicable).

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